# POSTURA®

Total Protection Ag+

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## Technical assistance systems for lying position postures

## POSTURA®

**TIPS**

The first prevention of the pressure ulcers to the bed consists in changing the points of support of the body on the mattress on average every three hours to limit the prolonged compression of the vessels to bony projections and to allow the irrigation of tissues. Do Not overtighten the bed linen thus reducing the effective support of the mattress.

**INSTALLATION**

See all possible configurations on the penultimate page.

**INDICATIONS :**

Positioning members (hips, knees, feet) lying down

**CONTRAINDICATIONS**

The circular modular cushion (VCP12CIC) has not been designed to accommodate a seated patient.

**COMPOSITION OF TECHNICAL AIDS:**

CUSHION: Cushion cover with CIC (cross-infection control) Ag+(Sanitized Ag) Expanded polystyrene microspheres (EPS), in which 99% of the volume is made up of air. Bacteriostatic polyester thread seams.

**CLEANING / DISINFECTION: • Cushions Total Protection Cover Ag+**

100% Promust CIC : 58 % polyester – 35 % polyurethane polycarbonate – 8% others

Item treated with a biocidal substance that is safe for the user. [www.winnicare.fr](http://www.winnicare.fr)

**CARE RELATING TO PROTECTIONS**

- Direct contact with injured skin does not pose a risk to the patient: the material has the Class I Oeko-Tex® 100 seal of approval.
- Prohibit scouring, stripping or dissolving products and sharp objects in direct contact with the protective covering.
- Carry out a visual check once or several times a year to ensure that the cushions are in good condition. In the event that any modification to the aesthetics of the cushion or to the seams has been made, the product shall be withdrawn and destroyed.

**STORAGE**

• Technical assistance systems should preferably be stored flat, out of direct sunlight and excessive humidity.

**DISPOSAL**

• Do not burn or dump the support in outside areas intended for this purpose.

**GARANTIE**

Technical assistance systems carry a 2 years guarantee, under normal conditions of use. This guarantee begins on the date the product is purchased from your dealer. This guarantee does not cover normal wear on the product and its protective covering and does not substitute legal guarantees. Please contact our distributor, identifying the offending product by stipulating the product lot number (See printed markings on the protection surface). They will carry out the necessary procedures with WINNCARE Services (04 66 02 15 23) in order to carry out repairs or make an exchange.

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### LIMPEZA / DESINFECÇÃO: • TOTAL PROTECTION DEVICES AG +