	<b>Manuel Utilisateur</b>			
	 VTAC410/CIC VTAC450/CIC	 VDT41A/CIC VDT45A/CIC		
 			 * P A 0 0 1 0 1 1 5 *	MAJ le 26/09/2017

**PROPRIÉTÉ**

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 	<b>User Manual</b>			
	 VTAC410/CIC VTAC450/CIC	 VDT41A/CIC VDT45A/CIC		

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**RECOMMENDATION**

The first thing to do in order to prevent pressure ulcers for someone lying down or in a seated position is to change the support's pressure points against the body every three hours (on average) in order to limit prolonged compression against the vessels close to overhanging bones and to allow the tissue to be irrigated.

**INDICATIONS**

Foot-raising splint: Helps prevent and/or treat pressure ulcers in high-risk situations or to correct extremely poor posture. Non-vigilant condition (paralysis of one side, medullary affliction, coma...).

Paralysis or paresis of ankle raising muscles as well as related sensorial disorders. Effects of severe vascular afflictions on lower limb. Effects of heavy orthopedic surgery entailing fixation of the lower limb. Discharge of pressure on the heels: Total or quasi-total loss of mobility of lower limbs with no risk of talipes equinus (surgery on lower limbs, loss of autonomy, etc.). Formation of a pressure ulcer on the heel.

**PRINCIPLE OF OPERATION**

VTAC410/CIC - VTAC450/CIC : The foot applies pressure at the metatarsus level in order to elevate or fix the foot at a right angle (100° flexing) without any lateral imbalance. The toes remain visible in order to observe their colour (cyanosis).

All references: Removal of heel pressure; the length of the cast frees malleolar areas in 90% of all cases.

Structures with flat surfaces facilitate the sideways removal of a lower limb without effort. Moderate reclining of the leg helps favour normal blood flow.

**PRECAUTIONS**

• Surveillance of pressure points several times a day and repositioning of the patient every 2 to 3 hours. To be used under medical prescription for cases of vascular insufficiency having an arterial origin.

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- Promust CIC+ PU-PVC Cover
- Promust PU HD Cover

						
Cover	Moderate wash at 90°C	Maximum authorised chlorine concentration 5000ppm	Do not iron	Do not dry clean	Drying with reduced thermal constraints	Use of an authorised surface detergent/disinfectant product

Item treated with a biocidal substance that is safe for the user. [www.winnocare.fr](http://www.winnocare.fr)

**SAFETY**

- Mobilise the leg every 2 to 3 hours and inspect the skin's condition (colour, warmth).
- Never expose the protection to direct contact with scouring/abrasive products, solvents or sharp/cutting objects.
- The product is not edible, do not ingest.

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- The support should preferably be stored horizontally, away from direct light and excessive humidity.

**GARANTEE**

This medical equipment carries a 1-year guarantee. This guarantee does not replace guarantees required by law. This guarantee takes effect on the date that the product is purchased from your distribution outlet. This guarantee doesn't cover regular wear of the product and its protection. Please contact your distributor and provide them with the defective product. The distributor will take the necessary steps with our company in order to proceed with either repair or a standard exchange after an assessment has been made.

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